

Saxon



Stannah

Some people travel across the world in search of the perfect landscape but thankfully, Gloria doesn't have to go too far. Now she has her Stannah stairlift, she can indulge herself with her favourite view of the park from her bedroom window, looking over the pond where she goes to feed the ducks.



Comfortable, convenient and easy to use, our Saxon stairlift is a popular choice for homes with narrow and straight stairs.

The chair

Joystick or push-button controls can be fitted on either arm, allowing you to travel up and down the stairs with ease and the swivel seat makes getting off at the top of the stairs safe and trouble-free. When you're not using the stairlift, the seat and arms fold away neatly, keeping your hallway and stairs clear for other users.

The rail

The Saxon is fitted to a strong rail which is available in a silver or bronze finish.

The rail is fitted to the stairs, not to the wall and installation is quick and clean, meaning minimum fuss and disruption in your home.

New and reconditioned models

Our Saxon chairs are available to buy as either new or reconditioned models. All pre-used stairlifts are reconditioned and rigorously tested by our team of engineers, using a 42 point check before being fitted in your home.

Our rental scheme

You can also rent our Saxon stairlifts which allows you to enjoy the benefits of a new or reconditioned Stannah, whilst paying for the stairlift one month at a time. We will take care of the servicing for you and remove it when you no longer need it.

The service

The reliability of our stairlifts comes from the quality of the engineering and design. For your added peace of mind, all our new and reconditioned products are backed up by a full warranty and service package. Just like any piece of machinery, we recommend that your stairlift is serviced annually by a trained engineer. We will arrange with you to come and visit to give the stairlift a thorough and complete service, putting it through its paces.

You won't have to wait for service or help as our nationwide teams of engineers are locally-based. They are on standby 24-hours a day, 365 days a year to respond quickly in the unlikely event that you might need assistance.

Self-charging battery

The Saxon always has the optimum amount of power so you get a smooth ride with a gentle start and stop. It even works if there is a power cut.



Keyswitch

Occasionally you might need to immobilise the stairlift. If children are around for example, you can use this key to make sure it can't be used.



Retractable seatbelt

Fasten the seatbelt for added security.

Adjustable seat height

We will adjust the height of the seat, so it's at the most comfortable setting for you, allowing you to get off the chair easily and without fuss.



Safety edges

If there is something blocking the chair on the stairs, the safety edges recognise this and the lift will stop automatically.



Wall control

You can use the wall controls to call the stairlift for yourself or send it to someone else.

Upholstery

You can choose from a range of colours to suit the look of your home.



Arm-to-footrest link

To store the stairlift away, you can fold the arm and the footrest folds away with it. There's no awkward bending down, which is particularly useful at the top of the stairs.



Directional controls

Right-handed or left-handed, we can put the controls on either arm. We can fit joystick or push-button controls, whichever you prefer.



Swivel seat

Using the levers enables you to swivel the seat easily, making it simple to get off the chair. Alternatively, you can choose to have a powered swivel seat, which turns the chair to face the landing automatically.



Straight rails

Fitted to the stairs and not to the wall, the straight rail is available in a silver or bronze finish.





1. Using the Saxon is simple and easy. To unfold it, pull down the seat, followed by the arm. The arm-to-footrest link will automatically lower the footrest down too.



2. Once unfolded, make yourself comfortable in the seat. Fasten the seatbelt for extra security.



3. By pressing gently on the controls in the direction you wish to go, you can travel up and down the stairs.



4. Once at the top of the stairs, you can swivel the seat to face the landing, allowing you to get off the chair in complete safety.



5. If you wish, you can leave the chair in its swivelled position, creating an effective barrier.



6. When you are not using the stairlift, it can be folded away so that it takes up less space and leaves the stairs clear for other users.

**Stannah contact details:
Watt Close, East Portway,
Andover, Hampshire
SP10 3SD**

**Tel: 0800 715358
Fax: 01264 338043
stannahstairlifts.co.uk**

Our company policy is one of continuing product improvement. We therefore reserve the right to alter the product specifications and to change and improve design at any time. Reconditioned products may have different specifications than those illustrated depending on availability. Full operating instructions are supplied for all products. Stannah is a registered trademark. The CE mark indicates that this product complies with the applicable European directives which relate to health, safety, environment and consumer protection. Stannah Stairlifts Limited has been accredited to ISO9001:2008 since 2009. An example of this product has been independently tested and approved. As with any other lift on whose reliability and safety you rely, we recommend that your Stannah stairlift is serviced annually by a trained engineer.



- **14 day money back guarantee**
- **Full warranty and service package**
- **24 hour, 365 day call out**
- **Nationwide, locally-based engineers**



The Stannah Promise

**Always true
to our word**

Stannah